

# Tecotherm Setup v.10-15-25


## 1. Attach power plug




## 2. Power on unit



## 3. Initial Start Up screen. Confirm alarms are audible.

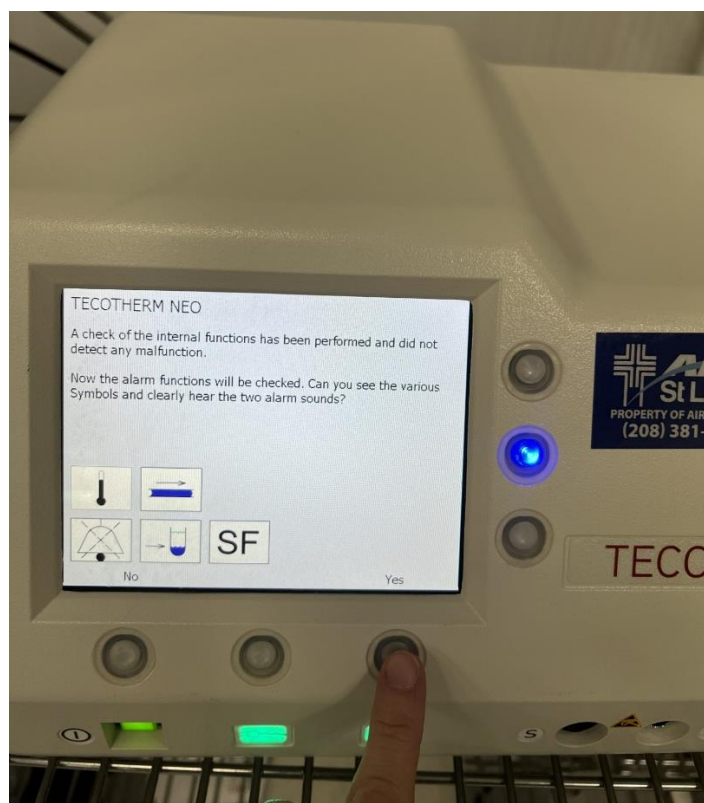


4. Select “Constant Mattress Temperature Mode” Press start to allow the pump to start introducing water.

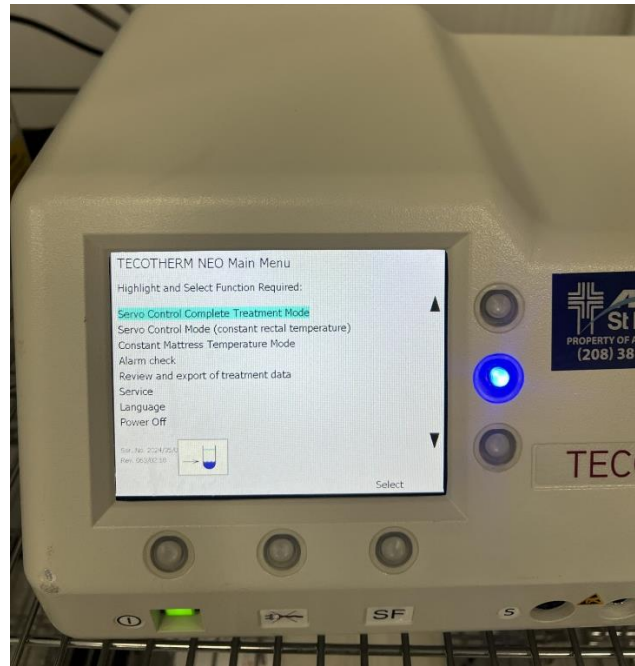


5. Fill the fill-up bottle with 450mL of Sterile Water. Then connect hoses to top ports. Attach hose to the cooling blanket and the bottom ports of the Tecotherm. Invert water bottle so the water will flow into the unit.




6. Fill the blanket until it feels like inverted bubble wrap. This may take a few minutes. Refill the fill-up bottle if needed.
7. Once the blanket is filled. Close the screen and select the options button-End Treatment and Save Data. Then press select to go back to the original screen.
8. Insert rectal temperature probe into the baby (5-6 cm). Connect rectal temperature cable to the front of the machine.

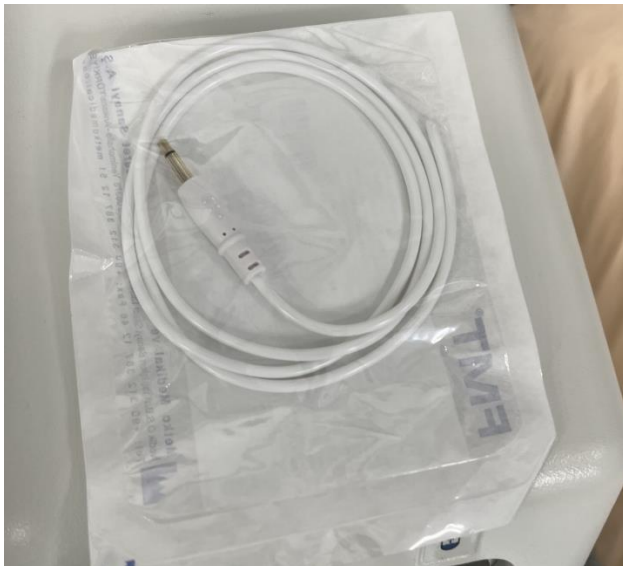

## 9. Select Mode: Servo Control Mode (constant rectal temperature)

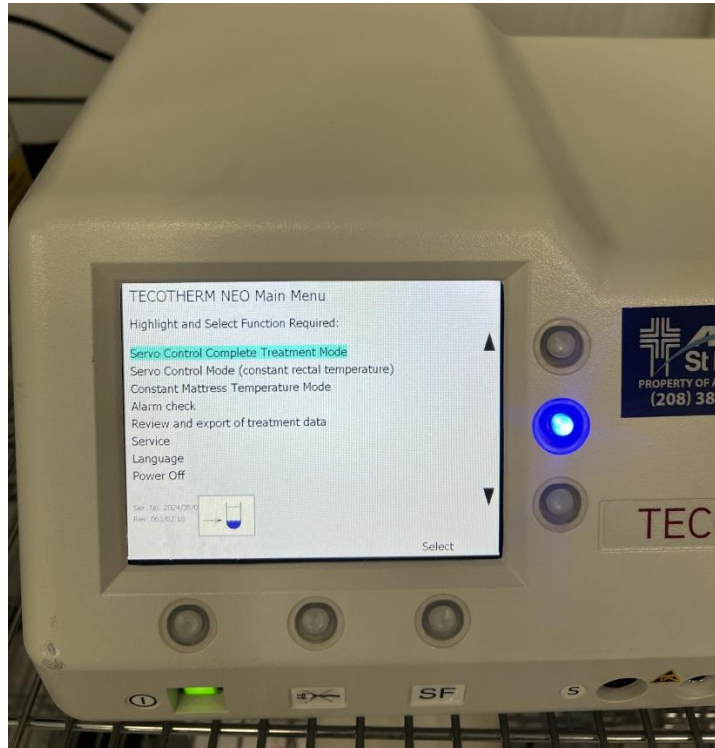

## 10. Confirm set point for mattress temperature. Confirm Time to final temperature.

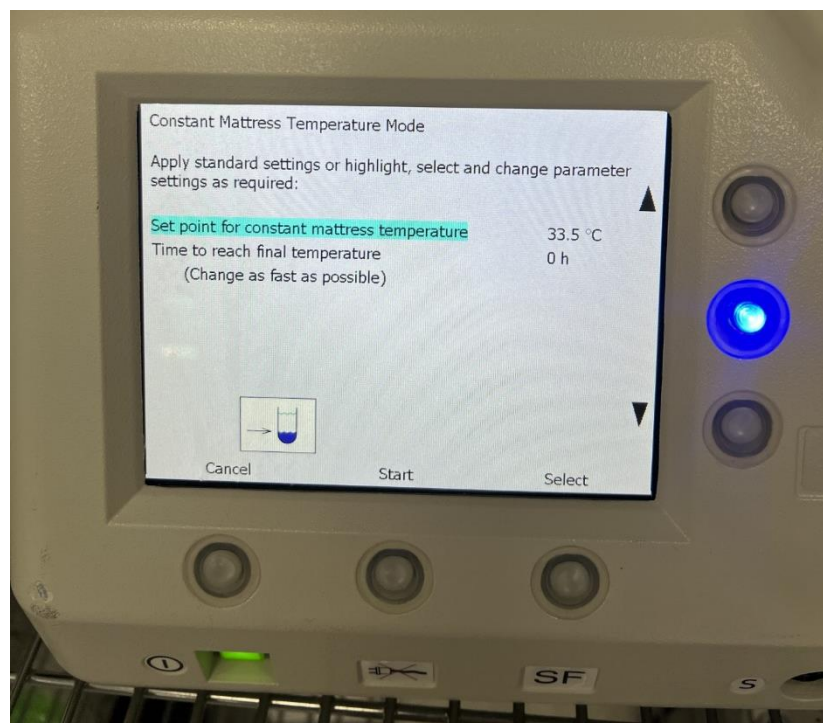

11. Continue to monitor blanket for need to refill.
12. To end treatment, select options-end treatment and save data. Hit select. Then scroll down and select “Power Off”.

